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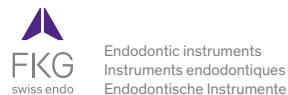
FKG REF. 99.000.10.02A_XX - notice n°101 - 2017/04

Sterilization protocol for reusable medical devices

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General advice:

Instruments stamped
are intended for single use only

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EN Sterilization protocol for reusable medical devices

Non sterile products

First use

Carefully follow the rules herebelow

Cleaning and Sterilization

Cleaning

1. Manual cleaning with/with-out Ultrasonic assistance

Thorough cleaning allows efficient disinfection & sterilization.

2. Rinse

Under distilled/demineralized water at least 1 min.

Then dry the instruments.

Sterilization

5. According EN/ISO 17664 protocol

Autoclave: 134°C/273°F,

2.2 bar during 18 min.

 Always refer to manufacturer's instructions for cycle & duration

Second and following uses

7. Pre-disinfection

Plunge instruments after use in detergent solution and eventually brush them manually

 Complete operating protocol and warnings on www.fkg.ch

General information

Pre-/Disinfection

DO NOT USE

solutions containing:

- phenol (corrosion)
- aldehyde (blood fixation)
- di-/trietanolamines (corrosion)

Caution:

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

A single patient use is recommended to avoid cross-contamination.

Sterile products

First use

1. Open the alveolus
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice

Second use:

Carefully follow the rules herebelow

Second and following uses.

3. Inspection - Check

Sort out damaged or worn out instruments.

4. Packing

Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

6. Storage

Keep devices in sterilization wrap/pouch in a dry and clean environment.

8. Rinse

Under running water at least 1 min.

Then dry the instruments.

9. Follow steps 1 - 6 above

Sterilization with chemiclav or dry air devices

have not been validated by FKG Dentaire.

Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

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