Instructions for use

- EN** see www.fkg.ch for additional information
FR informations supplémentaires sur www.fkg.ch
DE weitere Informationen auf www.fkg.ch

ScoutRace & ISO 10

Endodontic instruments
 Instruments endodontiques
 Endodontische Instrumente

FKG REF. 99.7AA.10.15A.XX - n°104 - 2018/01

ScoutRace & ISO 10

Description - Beschreibung

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- 21/16mm ● 25/16mm ● 31/16mm

EN Golden rules

- Speed: 600-800 rpm
 Torque: 1.5 Ncm
- Long back and forth gentle strokes
 - Light touch, let the instrument work
 - Work 3-4 seconds in a row, get out, clean the blade and irrigate the canal

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For canals with a severe apical curvature or S-shape

Manual scouting with ISO 06-08 SSt files up to estimated working length (WL); confirm WL with an apex locator

1. Introduce ScoutRace (SR) 10/.02 in rotation and reach WL;
2. Continue with SR 15/.02 up to WL;
3. Finish glide path with SR 20/.02 up to WL;
4. Final shaping with the main NiTi sequence (e.g. iRace or BioRace).

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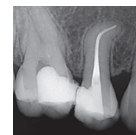
1. Pre-op: Pulpitis/Pulpite

2. Difficulty/Difficulté/Herausforderung

3. Post-op: 35/.04

Race ISO 10 1 blister includes/contient/enthält: 2 × 10/.02, 2 × 10/.04, 2 × 10/.06

For narrow and/or calcified canals

Manual scouting with ISO 06-08 SSt files up to estimated working length (WL); confirm WL with an apex locator.

1. Introduce Race 10/.02 in rotation and reach WL;
2. Continue with Race 10/.04 up to WL;
3. Finish glide path with Race 10/.06 up to WL;
4. Final shaping with the main NiTi sequence (e.g. iRace or BioRace).

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1. Pre-op: Pulpitis/Pulpite

2. Difficulty/Difficulté/Herausforderung

3. Post-op: MB & DB - 35/.04
 P - 50/.04

Cases: courtesy of / Cas: source / Fälle: Genehmigung von - Dr. G. Debelian (No)

EN Sterilization protocol for reusable medical devices

Non sterile products

First use

Carefully follow the rules herebelow

Cleaning and Sterilization

Cleaning

1. Manual cleaning with/without Ultrasonic assistance

Thorough cleaning allows efficient disinfection & sterilization.

2. Rinse

Under distilled/demineralized water at least 1 min.

Then dry the instruments.

Sterilization

5. According EN/ISO 17664 protocol

Autoclave: 134°C/273°F,

2.2 bar during 18 min.

 Always refer to manufacturer's instructions for cycle & duration

Second and following uses

7. Pre-disinfection

Plunge instruments after use in detergent solution and eventually brush them manually

 Complete operating protocol and warnings on www.fkg.ch

General information

Pre-/Disinfection

DO NOT USE

solutions containing:

- phenol (corrosion)
- aldehyde (blood fixation)
- di-/trietanolamines (corrosion)

Caution:

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

A single patient use is recommended to avoid cross-contamination.

Sterile products

First use

1. Open the alveolus
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice

Second use:

Carefully follow the rules herebelow

Second and following uses.

3. Inspection - Check

Sort out damaged or worn out instruments.

4. Packing

Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

6. Storage

Keep devices in sterilization wrap/pouch in a dry and clean environment.

8. Rinse

Under running water at least 1 min.

Then dry the instruments.

9. Follow steps 1 - 6 above

Sterilization with chemiclav or dry air devices

have not been validated by FKG Dentaire.

Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

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